


Go Knotless.

Speed of a running stitch.


Security of an interrupted stitch.

Value price of a traditional suture.

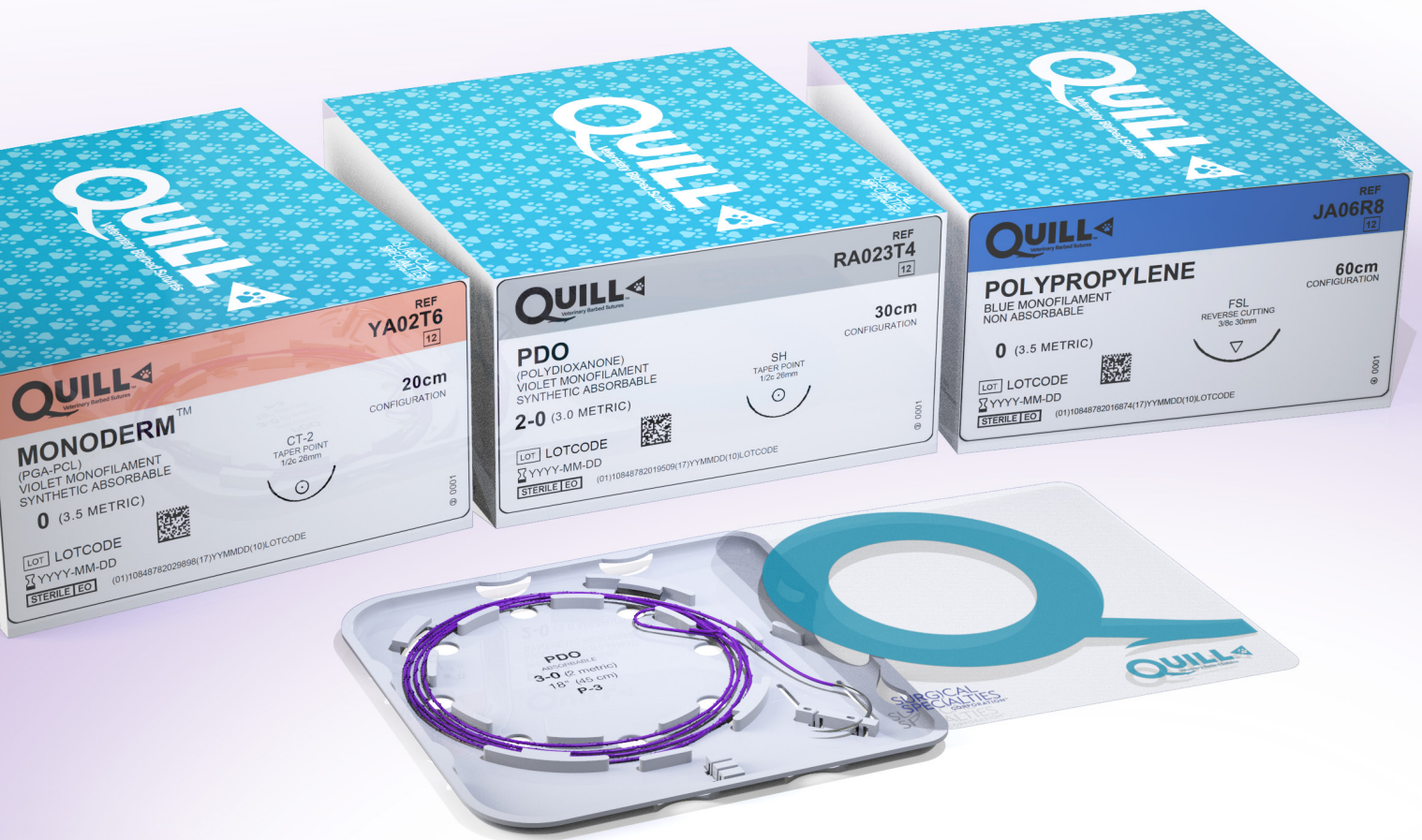

Create a better wound healing environment with a watertight close.

- ✓ **No knots to bury**, eliminating irritation caused by suture tags.
- ✓ **Intuitive, easy to use design** with no change to standard technique of running suture.

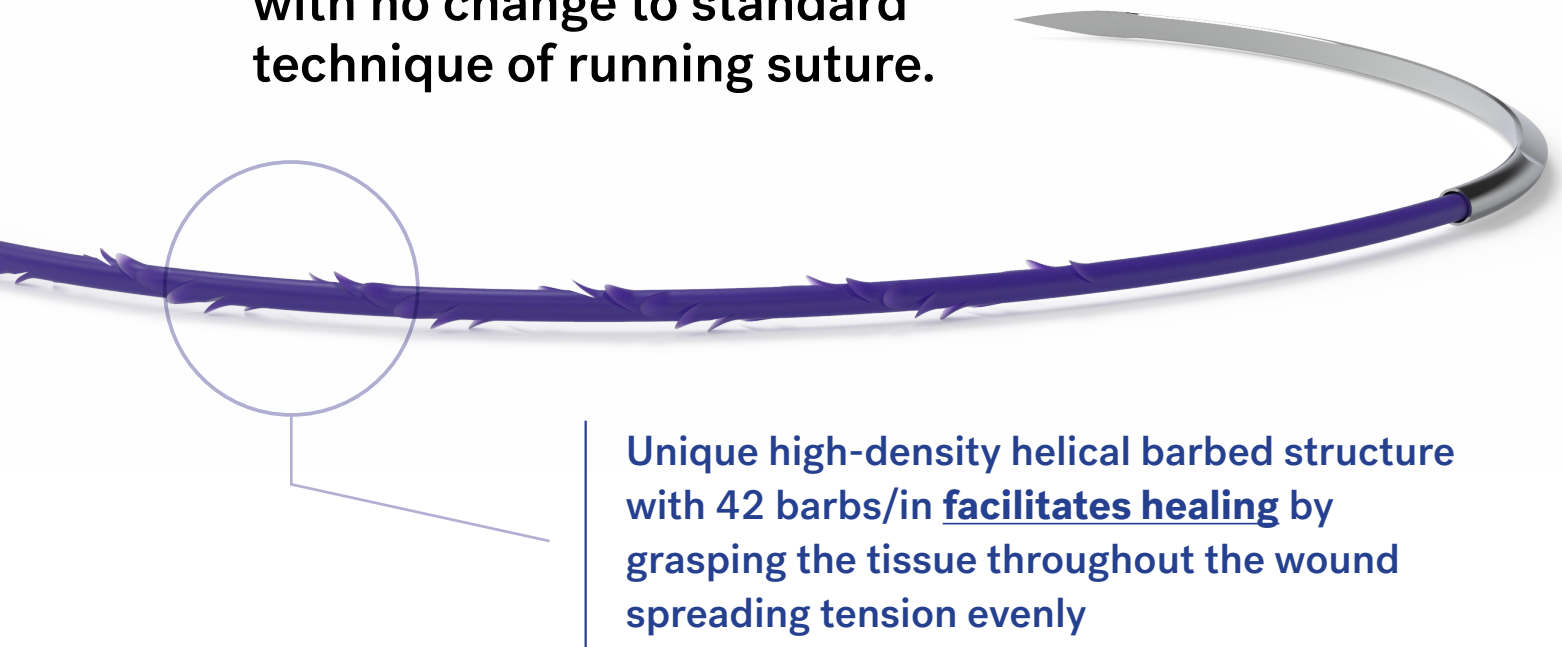

Atraumatic tissues passage




As much as a **50% reduction in closure times** has been reported with using Quill™ suture.

Closure Type



Uni-directional Barbed Suture

Single armed
Adjustable loop for anchoring

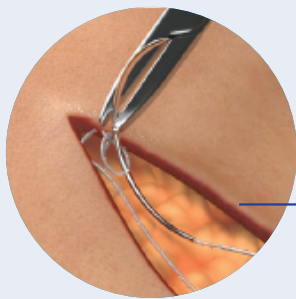

1 Pass first needle at apex. Then pass through variable loop.

2 Approximate tissue in a running fashion.

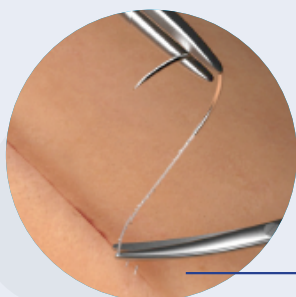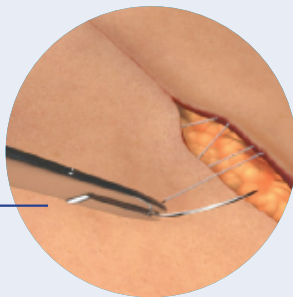


3 End with a backstitch or exiting lateral to end; cut off excess.

Closure Type



Bi-directional Barbed Suture

Double armed
Barbs reverse direction at midpoint

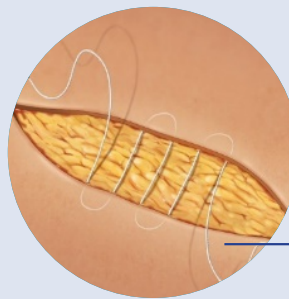

1 Take 2 bites with each needle, centering transition point of suture.

2 Cinch wound together.

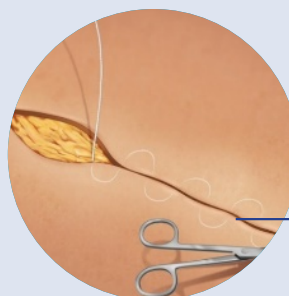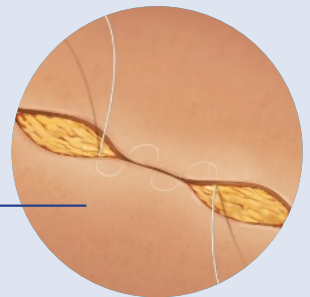

3 End with a backstitch or exiting lateral to end; cut off excess.

When choosing Quill barbed suture, upsize by one USP when converting from traditional suture and those of the competitors. For example, if previously using 3-0 USP traditional suture, choose 2-0 Quill suture.

Choose your material.

Offered in various materials and sizes fitted with premium needles.

Short-Term Absorbable Monoderm™ (PGA-PCL)

Monofilament

Dyed (Violet)

Essentially complete in 90-120 days

Long-Term Absorbable PDO (polydioxanone)

Monofilament

Dyed (Violet)

Essentially complete by 180 days

Non-Absorbable Polypropylene

Monofilament

Dyed (Blue)


Permanent

Choose your style and length.

Uni-directional Lengths



Bi-directional Lengths

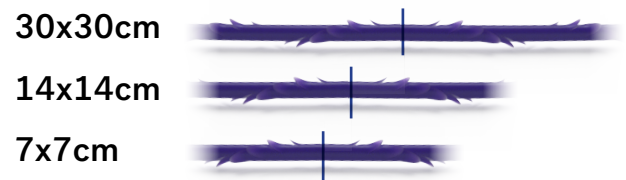

Choose your needle.

Reverse Cutting Needles ▽

TruStitch	Quill Vet	Ethicon™	Covidien™	
3/8 Circle				
FS-2*	FS-2	FS-2	C-13	19mm
FS-1*	FS-1	FS-1	C-14	24mm
FSL	FSL	FSL	C-16	30mm
FSLX*		FSLX	C-17	36mm
1/2 Circle				
CP-2*	CP-2	CP-2	GS-10	26mm
CP-1*	CP-1	CP-1	GS-11	36mm
CP*	CP	CP	GS-12	40mm

Taper Point Needles ●

TruStitch	Quill Vet	Ethicon™	Covidien™	
1/2 Circle				
RB-1*	RB-1	RB-1	CV-23	17mm
SH-1*	SH-1	SH-1	CV-25	22mm
SH*	SH	SH	V-20	26mm
CT-2*	CT-2	CT-2	GS-22	26mm
CT-1*	CT-1	CT-1	GS-21	36mm

**SURGICAL
SPECIALTIES**
CORPORATION™

Order Direct at 877-991-1110

For more information, visit SurgicalSpecialties.com

Surgical Specialties Corporation, 247 Station Drive, Suite NE1, Westwood, MA 02090, U.S.A.

Phone: 610.404.3520 • 877.991.1110 (USA and Canada only) • Fax: 610.404.4010

All trademarks are owned by the company and its affiliates. ©2019 Surgical Specialties Corporation. All Rights Reserved.