

Go Knotless.

From the world's largest independent suture manufacturer, Surgical Specialties

Speed of a running stitch.

Security of an interrupted stitch.

Value price of a traditional suture.

Quill™ Veterinary Variable Loop


From the makers of the original barbed suture comes an unparalleled unidirectional barbed configuration with a unique variable loop design that allows for easier deployment.

- ✓ **Better wound healing** with a watertight close.
- ✓ **No irritation** from knots or suture tags.
- ✓ Up to **50% quicker closures** are reported with Quill.




QUILL™
Veterinary Barbed Sutures


Basic technique for the Quill Variable Loop Device

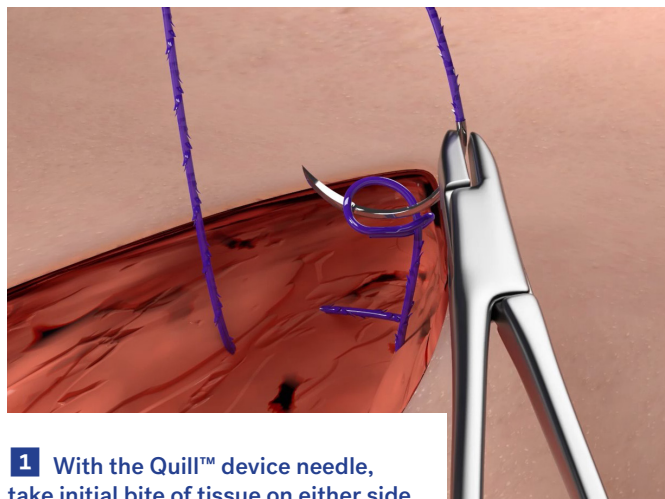


1 With the Quill™ device needle, take initial bite of tissue on either side of the apex of the wound. Then, pass the needle through the variable loop.

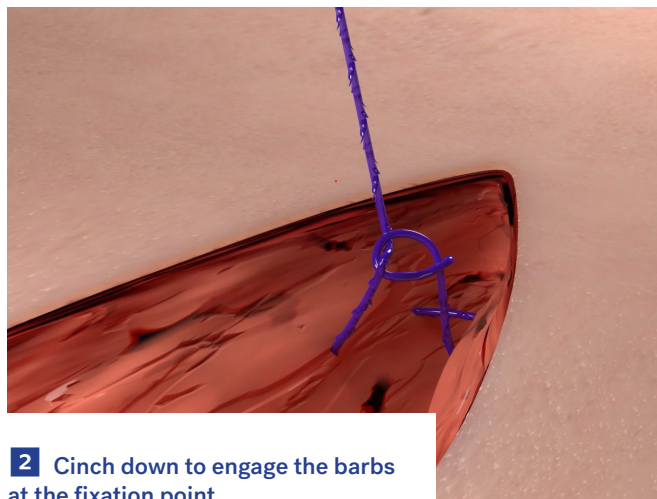

2 Cinch down to engage the barbs at the fixation point.

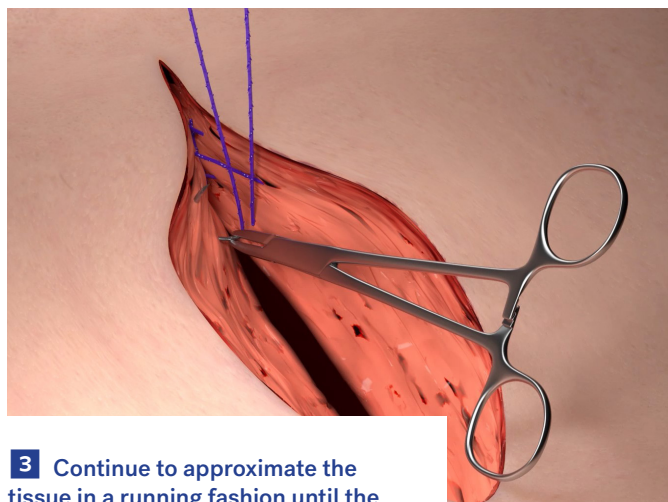

3 Continue to approximate the tissue in a running fashion until the opposite end of the wound is reached. Use care to approximate the tissue and not strangulate.

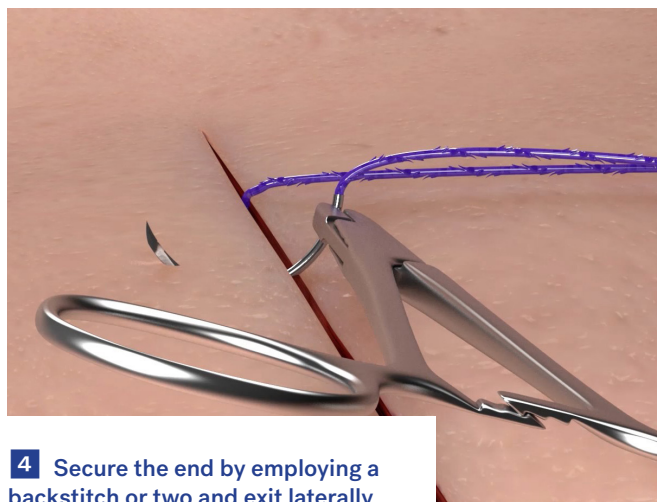

4 Secure the end by employing a backstitch or two and exit laterally and cut flush with the skin once approximation is complete.

How to get most benefit from Quill Veterinary Suture:

- ✓ **Handle the material gently.** Complete 3 or 4 loose throws before pulling to approximate the tissue.
- ✓ We recommend **4-6 trials** to master the technique.
- ✓ Quill suture is determined by its outer diameter, so we recommend **upsizing by one USP** compared to traditional suture.

**SURGICAL
SPECIALTIES**
CORPORATION™

SSC-103 R0

Order Direct at 877-991-1110

For more information, visit [SurgicalSpecialties.com/vet-brands](https://www.SurgicalSpecialties.com/vet-brands)

Surgical Specialties Corporation, 247 Station Drive, Suite NE1, Westwood, MA 02090, U.S.A.

Phone: 610.404.3520 • 877.991.1110 (USA and Canada only) • Fax: 610.404.4010

All trademarks are owned by the company and its affiliates. ©2020 Surgical Specialties Corporation. All Rights Reserved.